

 **LAKE EFFECT**



MICROBECARE™

Microbe Safety Mask™

Treated with MicrobeCare™

Instructions for Use

READ INSTRUCTIONS BEFORE USE

Made in the USA

WARNING: THIS PRODUCT IS NOT A NIOSH (CDC) CERTIFIED N95 RESPIRATOR OR AN FDA APPROVED (CLASS II) SURGICAL MASK.

The CDC has recognized where N95 respirators and surgical masks are not available, as a last resort, it may be necessary for healthcare professionals to use masks that have never been evaluated or approved by NIOSH or homemade masks. Caution should be exercised when considering the use of non-NIOSH approved masks or homemade masks, like this product. Use of this product will not eliminate the risk of illness, and does not remove the risk of contracting any disease, illness, or infection. If you experience difficulty breathing, pain, swelling, changes in sensation or unusual reactions, discontinue the use of this product immediately and seek medical attention.

CARE INSTRUCTIONS: Hand wash only using warm water (NOT HOT) and a mild detergent. Rinse thoroughly. Lay flat to dry. Do not machine wash or machine dry.

INSTRUCTIONS FOR USE

1. Hold the mask over your nose and chin, fuzzy side out.
2. Pull the ear straps over each ear.
3. Mask fit should be snug, yet comfortable.
4. Adjust sizing as needed by removing the Velcro® earpiece portion.
5. For a more snug fit, move the ear pieces closer together. Note the ear pieces can be angled.
4. Trim excess fabric if necessary.



These directions are guidelines only and not offered as medical recommendations. US and foreign patents pending.

About MicrobeCare™: Each mask is treated with MicrobeCare™, which provides a 99.9% reduction in surface contamination, creating a barrier against a broad spectrum of bacteria and viruses. Effective and proven, MicrobeCare is the only EPA registered antimicrobial in the US.

Product Numbers

Product #	Description	Quantity
13483-00-10	Microbe Safety Mask	10
13483-00-50	Microbe Safety Mask	50

View additional videos at lakeeffectmed.com